

Sendoline®

PERFECT ENDO



S5

ROTARY SYSTEM



S5 ROTARY SYSTEM

S5 Endo Motor

SAFE, SIMPLE AND EFFICIENT!

Sendoline's innovative system for mechanical cleaning. Simply perfect endo!

Features:

- 1. Easy to use**
S5 Endo Motor + S5 Rotary Files = S5 Rotary System
Uses only 5 settings with 5 files (S5 Rotary Files)
Complete versatility, works with most rotary file systems
Cordless freedom: no cables
- 2. Security**
2 options: auto reverse / auto stop
Torque indicator: Light indicates when 75% of the torque is reached
Low speed: 300 rpm
- 3. Reduced treatment times**
Treatment can be completed safely, and more efficiently using the S5 Rotary System with the unique S-Profile
- 4. Accessibility**
Only 11mm diameter contra angle head
Shorter shanks (13mm) on rotary files
- 5. Non-stop**
2,5 hrs continuous use
- 6. Ergonomics**
Weight: 120g
- 7. Apex Clip**
Easy connection to an apex locator

Apex Clip
Now with Apex Clip for easy connection to an apex locator.

Safe and secure
Light indicates when 75% of the torque is reached

5 settings = 5 files



S5 Rotary Files



1, 2, 3, 4, 5 – FIVE STEPS TO PERFECT ENDO.

The Rotary System from Sendoline offers a simple and effective solution for the dentist.

Features:

1. Only 5 files and all with the unique S-profile
2. Shorter shank (13mm) for better access
3. Long progressive flutes to prevent the "screw-in-effect"
4. Enhanced debris transportation
5. Developed for use with the S5 Endo Motor

"The S5 file system is so simple to use, it gives me predictable results every time!" *Dr. Paul Godbold BDS General practitioner, UK*

"The S5 rotary system helps me to efficiently and safely create a canal shape that is perfect for cleaning, disinfecting and then obturating every canal I come across."
Dr. Claire Redley BDS General Practitioner, UK

S5 Apex Locator

A highly accurate electronic apex locator which allows precise measurement of the root canal's working length. Its compact design makes it portable and easy to use.

Features:

1. High precision of apical foramen localization in wet and dry canals
2. Fully automatic measurements
3. Compact and user-friendly design
4. Automatic turn-off
5. Audio feedback with volume control



Preparation

Big Apical Files

How do I shape canals apices larger than a size 30?

For the best results in cases of teeth with larger apical diameters, the Big Apical Files are the perfect completion for the S5 Rotary System. Shape the canal as usual with the S5 Rotary System and simply finish the apical portion using the Big Apical Files.

Available in sizes: TAPER/ISO 04/35 - 04/50 and file length 23 or 28 mm.



Obturation

S5 GP Points

How do I obturate canals prepared with the S5 Rotary System?

Sendoline's S5 GP Points are specifically designed to match the canal shape created with the S5 Rotary Files. Finish the preparation with either file no. 2 or no. 3 and simply select the S5 GP Point that corresponds to that file. Package: 60 pcs/box

S5 Paper Points

The Sendoline S5 Paper Points, which are rolled to exact dimensions, simplify the root canal treatment with their excellent absorbance ability and they work perfectly with the S5 GP Points.

Available in sizes: Taper / ISO: 2 (06/30) och 3 (04/30).
Package: 60 pcs/box



Retreatment

S5 Retreatment Files

The S5 Retreatment Files enables the removal of old filling materials quickly and easily. To conduct the retreatment, simply apply the Crown Down technique using only 4 files. The markings of the S5 Retreatment Files correspond to the programs of the S5 Endo Motor. The S5 Retreatment Files are available in 23 mm length. TAPER/ISO file 1: 12/30 and file 2-4: 08/30, 06/30 and 04/30.

